INFORMATION FOR USE, READ CAREFULLY
TEAVE KASUTAMISEKS, LUGEDA TÄHELEPANELIKULT
UZMANĪGI IZLASIET LIETOŠANAS INSTRUKCIJU
INFORMACIJA APIE NAUDOJIMĀ. ATIDŽIAI PERSKAITYKITE

EN

HemaGel®

INTENDED USE: Hydrophilic gel for the treatment of surface skin injuries.

WHY SHOULD HemaGel® BE USED FOR WOUND TREATMENT?

- chemically binds oxygen radicals, which arise in the course of the process of healing wounds
- covers the wound, thus protecting it against secondary infection from outside
- creates an elastic mechanical protective layer on the wound
- creates a suitable environment for granulations, including reducing secretion

COMPOSITION: HemaGel® is a hydrophilic gel composed of 2-hydroxyethylmetacrylate copolymer, which contains polymerically bonded sterically shielded amines (HEMA-HAS), macrogol and purified water (aqua purificata). HemaGel® has been developed by the Institute of Macromolecular Chemistry of the Czech Academy of Sciences.

HOW TO PROCEED WHEN USING THE GEL:

- First clean the wound from mechanical impurities, such as dust particles and remnants of previously used curative materials, which might subsequently irritate and damage the wound.
- Spread a thin layer of the gel (about 1 mm) over the entire area of the wound base, slightly exceeding to healthy skin. The gel will adhere to the wound base and will gradually absorb secretions and other fluids pouring out of the wound.
- Cover the wound with a non-absorptive material (such as greasy tulle available in pharmacies) and subsequently fix the wound with bandage or gauze square and a sticking plaster. After the gel being applied, cover small injuries with a cushioned adhesive plaster.
- The effect of the gel manifests itself with bulging. The unique gel technology prevents adherence to the wound, thus enabling easy and painless removal by mild wiping or rinsing.
- Change the bandage as necessary, every 12 – 48 hours, always depending on the condition of the wound and soaking through secondary wound covering or as instructed by the treating doctor.

TARGET GROUP AND LIMITATIONS OF USE:

The product is intended for adult patients.

Pregnancy and lactation: By reason of absence of clinical trials related to use in pregnant and breastfeeding women, HemaGel® can only be used after consulting your doctor.

Use in children: By reason of absence of clinical trials related to use in children, HemaGel® can only be used after consulting your doctor. In the event that slight skin irritation appears (such as slight itching or burning) in people with increased sensitivity or after application to sensitive areas (such as around the eyes), stop treatment and contact your doctor.

WARNINGS: In the event that side undesirable effects appear (such as increased pain, erythema and suchlike), discontinue treatment and contact your treating doctor. Keep out of the reach of children. For external use only. Do not use internally. Do not use after the expiration of the period indicated on the tube. An unused or expired product to be given back to the pharmacy. Use in children, pregnant and breastfeeding women is recommended after consultations with a doctor. Apply the product to the wound for a maximum period of 30 days. In case of use exceeding 30 days, consult your doctor. Close the tube carefully after every use.

STORAGE: Store in a dry place at a temperature of 15–25 °C.

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